

The Mandibular Stabilizer

- Keeps functional appliances properly seated during sleep.
- Applies only vertical force (NO maxillary retraction).
- Maintains proper lip closure and nasal breathing with or without an appliance in place.
- Adjustable for optimum comfort.

A new and exciting concept in maintaining the mandible in a closed, comfortable position...

The Comfort Support System is an extremely valuable aid in assisting proper facial growth by stabilizing the mandible (during functional appliance therapy), preventing it from dropping freely.

This system was developed to assist the patient in keeping an appliance in place while sleeping. Bionators and other orthodontic correctors often need this assistance in order to achieve optimum results.

The stabilizer has also proven to be effective for the maintenance of the mandible for patients with a compromised dentition while wearing their snoring/sleep apnea devices.

In addition, for patients who are diagnosed with sleep apnea and are undergoing continuous positive airway pressure (CPAP) therapy, the stabilizer maintains the mandible in a closed position, preventing the lower jaw from dropping freely, so that air will be contained during sleep.



Great Lakes Orthodontics, Ltd.
200 Cooper Avenue, P.O. Box 5111
Tonawanda, NY 14151-5111

800-828-7626

Local: 716-871-1161

Fax: 716-871-0550

Force exerted in a vertical direction.



Available in Red (155-028) and Blue (155-029)

Instructions for use:

- 1) Have patient place the stabilizer so that the chin is resting comfortably, as shown in the photo. Snap together at crown.
- 2) Secure horizontal elastic strap to stabilizer from ear to ear.
- 3) Attach vertical elastic strap from the top of the stabilizer to the horizontal strap.