

(300-306)
 Small
 (300-307)
 Large



Face Mask Headgear

- Easy to Fit
- Comfortable headstrap

(300-308)
 Small
 (300-309)
 Large



Hickham Headgear

- Comfortable for use while sleeping
- Does not inhibit mandibular movement
- Ideal for day-time wear

Nola protraction headgear are designed to bring individual teeth, groups of teeth or entire arches forward. The headgear is fully adjustable and offers two chin cup sizes.

Treatment Suggestions

Elastics attached to the terminal teeth will act like Class III elastics to the upper arch. In other words, they will tend to open the bite by extruding these teeth. It is therefore desirable to have the elastics attached to the anterior part of the arch. This is especially true in Class III cases where the premaxilla needs to be developed down and forward in a normal growth pattern.

Occasionally, the patient will perspire excessively for the first

three days the chin cup is worn. The tissue will adapt and the perspiring usually stops. Do not try to remedy the problem by drilling holes in the cup for air circulation. Air will not circulate through the holes and the patient will not like the spotted chin appearance when the cup is removed. If perspiring persists or the patient is allergic to acrylic, cloth liners are available.

ORDERING INFO ON BACK 



Great Lakes Orthodontics, Ltd.
 200 Cooper Avenue, P.O. Box 5111
 Tonawanda, New York, 14151-5111

800-828-7626 Fax: 716-871-0550 info@greatlakesortho.com



Orthodontic Service Landau
 Metzendorger Weg 25
 21224 Rosengarten 6
 Germany

Item	Order No.
Protraction Headgear & Replacement Parts	
Small Face Mask	300-306
Large Face Mask	300-307
Small Hickham	300-308
Large Hickham	300-309
Headstrap for Hickham (both sizes)	300-312
Forehead strap and lock nuts for Face Mask (both sizes)	300-313
Lock nuts for forehead strap (2/pk)	300-314
Disposable chin liners (10/pk)	300-315
Plastic protector tips (10/pk)	300-316

IMPORTANT REMINDER FOR DOCTORS!

To avoid risk of eye injury, be sure to inform patients and parents about proper safety precautions when placing and removing headgear. Also caution patients against rough or risky play while headgear is in place.



Great Lakes Orthodontics, Ltd.
 200 Cooper Avenue, P.O. Box 5111
 Tonawanda, New York, 14151-5111

To Order:
800-828-7626 Fax: 716-871-0550
info@greatlakesortho.com