

Packing Work Models & Bite Registrations for the Laboratory


It is essential to carefully pack models and bite registrations to avoid damage during shipping which could result in fabrication and treatment delays. Great Lakes designed new packing material to ensure that your models and bites arrive safely at our Laboratory.

Contact us for a catalog of all our Laboratory appliances and services, free shipping supplies, or to speak with a Laboratory Customer Service representative.

1.800.828.7626

Fax: 716.871.0550

www.greatlakesortho.com

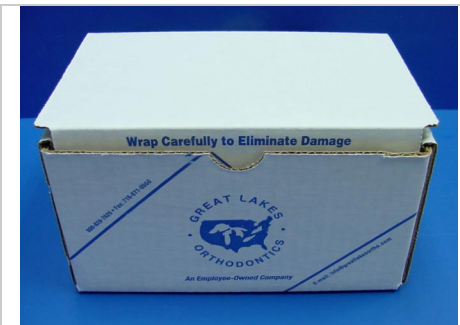

Shipping supplies include:

- 1 - Shipping box
- 2 - Foam sheets (top and bottom)
- 1 - Custom foam insert
- 3 - Mailing labels

Great Lakes shipping box and new packing materials will arrive assembled. Remove the top foam sheet to pack the models and bite registration.

Place bite registration in the bite cut out. Place the models in the right and left model cut outs with the arch facing up. Place the completed prescription form on top of the foam insert and replace the top foam sheet.

Tape the box lid closed, apply the pre-paid mailing label, and the package is ready to ship. There should be no movement in the box during shipping.

Helpful tips:

- Stone models are preferred
- Write the patient's name on the model base or mounting
- Complete the Rx and include it in the shipping box
- To avoid damage in transit, Great Lakes does not recommend sending a bitefork

Contact Great Lakes Technical Support Specialist at 716-319-1223 to request new tools including:

- Impression-Taking and Work Model Instructions
- Lucia Jig Centric Relation Bite Registration Guide
- Great Lakes Splint Appliance Selection Guide