smart moves.

predictability index





»»» predictability by malocclusion

	WITH CLEAR ALIGNER TREATMENT			PRE-ALIGNER OPTIONS TO TREAT LESS PREDICTABLE CASES	
TYPE OF MALOCCLUSION	VERY PREDICTABLE	MODERATELY PREDICTABLE	UNPREDICTABLE	PRE-ALIGNER STRATEGY	IMPACT ON SUBSEQUENT ALIGNER SEQUENCE LENGTH
CROWDING	0-3mm	3-6mm	6mm+	Removable pre-aligner	10-20% less aligners than with clear aligner therapy only 20-30% less aligners
				Fixed pre-aligner	than with clear aligner therapy only
SPACING	0-3mm	3-6mm	6mm+	TADs	20-30% less aligners than with clear aligner therapy only
OVERJET	0-3mm	3-5mm	5mm+	Pre-aligners or TADs	20-30% less aligners than with clear aligner therapy only
OVERBITE	0-2mm	2-4mm	4mm+	Pre-aligners or TADs	20-30% less aligners than with clear aligner therapy only
OPEN BITE	0-1mm	1-2mm	2mm+	TADs	20-30% less aligners than with clear aligner therapy only
MIDLINE	0-2mm	2-4mm	4mm+	Pre-aligners	20-30% less aligners than with clear aligner therapy only
				TADs	20-40% less aligners than with clear aligner therapy only

»»» predictability by tooth movement

MOVEMENT TYPE	тоотн	VERY PREDICTABLE	MODERATELY PREDICTABLE	UNPREDICTABLE
FACIAL-LINGUAL TIPPING	Anterior	0-5°	5-15°	15°+
	Posterior	0-5°	5-15°	15°+
MESIAL-DISTAL TIPPING	Anterior	0-5°	5-10°	10°+
	Posterior	0-5°	5-10°	10°+
TOPOUE	Anterior	0-5°	5-10°	10°+
TORQUE	Posterior	0-3°	3-6°	6°+
	Incisors	0-15°	15-30°	30°+
ROTATION	Canines	0-10°	10-15°	15°+
ROTATION	Premolars	0-5°	5-15°	15°+
	Molars	0-5°	5-10°	10°+
TRANSLATION	Anterior/Posterior	0-2mm	2-3mm	3mm+
	Incisors	0-1mm	1-2m	2mm+
INTRUSION	Premolars	05mm	.5-1mm	1mm+
	Molars	05mm	.5-1mm	1mm+
	Incisors	0-1mm	1-2mm	2mm+
EXTRUSION	Premolars	05mm	.5-1mm	1mm+
	Molars	05mm	.5-1mm	1mm+

»»» predictability of arch form adjustment

In this chart, you'll see the five categories of arch form adjustment. The overall movement within an arch is dictated by the teeth and alveolar bone within the arch. Every one of your patients are uniquely different, and can display varying results.

MOVEMENT TYPE	тоотн	VERY PREDICTABLE	MODERATELY PREDICTABLE	UNPREDICTABLE
	Distalize	0-2mm	2-4mm	4mm+
	Mesialize	0-2mm	2-4mm	4mm+
ARCH FORM ADJUSTMENT	Lingualize	0-2mm	2-3mm	3mm+
	Anteriors Forward	0-2mm	2-3mm	3mm+
	Width Expansion	0-2mm	2-4mm	4mm+

800.828.7626 > smartmovesaligners.com

